

BARD[®]

Cardiovascular Fabrics, Surgical Felts, Pledgets and Tapes

Proven to be Successful in Surgery



BARD FULL LINE OF CARDIOVASCULAR FABRICS...

...offers the surgeon a choice of synthetic patch materials which have been proven to be successful in surgery for the repair of septal defects and for patch grafting.

CARDIOVASCULAR FABRICS

| Product/Indications | Product Code | Size (cm) | Nominal Thickness (mm) | Size (inches) | Units/Box |
|--|--------------|-------------|------------------------|---------------|-----------|
| Knitted Polyester | | | | | |
| BARD® SAUVAGE® Filamentous Fabric Recommended for patch graft angioplasty such as in carotid endarterectomy procedures and ASD/VSD repairs | 007940 | 1.0 x 15.2 | 0.61 | 3/8 x 6 | 10 |
| | 007942 | 2.5 x 10.2 | 0.61 | 1 x 4 | 10 |
| | 007943 | 5.1 x 5.1 | 0.61 | 2 x 2 | 10 |
| | 007944 | 5.1 x 10.2 | 0.61 | 2 x 4 | 10 |
| | 007828 | 10.2 x 10.2 | 0.61 | 4 x 4 | 5 |
| | 007829 | 15.2 x 15.2 | 0.61 | 6 x 6 | 5 |
| BARD® DEBAKEY® Double Velour Fabric Used for patch graft angioplasty and repair of intracardiac defects | 007937 | 2.5 x 10.2 | 1.50 | 1 x 4 | 10 |
| | 007939 | 5.1 x 10.2 | 1.50 | 2 x 4 | 10 |
| | 007826 | 10.2 x 10.2 | 1.50 | 4 x 4 | 5 |
| | 007827 | 15.2 x 15.2 | 1.50 | 6 x 6 | 5 |
| BARD® DEBAKEY® Elastic Knit Fabric Used for patch graft angioplasty and repair of intracardiac defects | 007830 | 10.2 x 10.2 | 0.57 | 4 x 4 | 5 |
| | 007831 | 15.2 x 15.2 | 0.57 | 6 x 6 | 5 |
| Woven Polyester | | | | | |
| BARD® DEBAKEY® Woven Fabric A thin, lower permeability fabric for use in outflow tract repairs, patch graft angioplasty, and septal defects | 007957 | 5.1 x 10.2 | 0.28 | 2 x 4 | 10 |
| | 007955 | 10.2 x 10.2 | 0.28 | 4 x 4 | 5 |
| | 007956 | 15.2 x 15.2 | 0.28 | 6 x 6 | 5 |
| Knitted PTFE | | | | | |
| BARD® Edwards Outflow Tract Fabric Lower permeability material used for aortic and pulmonary outflow tract patching and intracardiac defects | 007979 | 5.1 x 10.2 | 0.91 | 2 x 4 | 10 |
| | 007834 | 10.2 x 10.2 | 0.91 | 4 x 4 | 5 |
| | 007835 | 15.2 x 15.2 | 0.91 | 6 x 6 | 5 |

BRAIDED TAPES

| Product/Indications | Product Code | Size (mm x cm) | Size (inches) | Units/Box |
|-------------------------------------|--------------|----------------|---------------|-----------|
| BARD® PTFE Braided Tape | 007913 | 2.0 x 61.0 | 1/16 x 24 | 20 |
| | 007914 | 4.0 x 61.0 | 3/16 x 24 | 20 |
| | 007917 | 2.0 x 91.4 | 1/16 x 36 | 20 |
| | 007918 | 4.0 x 91.4 | 3/16 x 36 | 20 |
| BARD® Polyester Braided Tape | 007915 | 3.0 x 61.0 | 1/8 x 24 | 20 |
| | *007916 | 3.0 x 91.4 | 1/8 x 36 | 10 |

For the purpose of tissue approximation, as a temporary or permanent ligature, and to retract vessels during a surgical procedure. BARD® PTFE Braided Tapes are packed 2 per sterile pouch, 10 pouches per box, unless otherwise noted.

*One Per Sterile Pouch.








BARD SELECTION OF SURGICAL FELTS...

...gives the surgeon several options in material thickness and permeability. These felts are used in various ways in cardiac surgery, most commonly as a patch, a buttress for sutures, and a material used in procedures that reinforce large segments of the left ventricular myocardium.

SURGICAL FELTS

| Product/Indications | Product Code | Size (cm) | Nominal Thickness (mm) | Size (inches) | Units/Box |
|--|--|-------------|------------------------|---------------|-----------|
| BARD® Polyester Felt Used as a buttress for sutures and friable tissue | 008972 | 15.2 x 15.2 | 1.65 | 6 x 6 | 5 |
| | 007021 | 2.5 x 2.5 | 1.85 | 1 x 1 | 5 |
| BARD® Low Porosity PTFE Felt Low porosity helps to control bleeding | 007838 | 10.2 x 10.2 | 1.85 | 4 x 4 | 5 |
| | 007839 | 15.2 x 15.2 | 1.85 | 6 x 6 | 5 |
| | 007974 | 0.6 x 5.1 | 1.65 | 1/4 x 2 | 10 |
| BARD® PTFE Felt Used as a buttress for sutures and friable tissue | 007975 | 1.3 x 10.2 | 1.65 | 1/2 x 4 | 10 |
| | 007973 | 2.5 x 2.5 | 1.65 | 1 x 1 | 10 |
| | 007968 | 2.5 x 10.2 | 1.65 | 1 x 4 | 10 |
| | 007976 | 2.5 x 15.2 | 1.65 | 1 x 6 | 10 |
| | 007977 | 5.1 x 5.1 | 1.65 | 2 x 2 | 10 |
| | 007836 | 10.2 x 10.2 | 1.65 | 4 x 4 | 5 |
| | 007837 | 15.2 x 15.2 | 1.65 | 6 x 6 | 5 |
| | BARD® PTFE Felt (Thick) Extra thickness provides added support to tissue | 007018 | 1.0 x 15.2 | 2.87 | 3/8 x 6 |
| 007019 | | 2.5 x 2.5 | 2.87 | 1 x 1 | 5 |
| 007958 | | 10.2 x 10.2 | 2.87 | 4 x 4 | 5 |
| 007959 | | 15.2 x 15.2 | 2.87 | 6 x 6 | 5 |

PTFE FELT PLEDGETS

| Product/Indications | Product Code | Size (mm) | Nominal Thickness (mm) | Size (inches) | Units/Box | Shape | Size Guide (1:1) |
|--|--------------|-----------|------------------------|---------------|-----------|-----------|---|
| BARD® PTFE Felt Pledgets Used as buttresses under sutures when there is a possibility of sutures tearing through tissue. These pledgets are used in various surgical suturing procedures, such as vascular closure, septal repair, myocardial closure, and valvular suturing | 007970 | 4.8 x 6.0 | 1.65 | 3/16 x 1/4 | 250 | Rectangle |  |
| | 007963 | 4.8 x 9.5 | 1.65 | 3/16 x 3/8 | 250 | Rectangle |  |
| | 007972 | 7.9 x 7.9 | 1.65 | 5/16 x 5/16 | 250 | Square |  |
| | 007965 | 6.0 x 6.0 | 1.65 | 1/4 x 1/4 | 250 | Square |  |
| | 007969 | 4.8 x 6.0 | 1.65 | 3/16 x 1/4 | 250 | Oval |  |
| | 007984 | 4.8 | 1.65 | 3/16 | 250 | Round |  |
| BARD® PTFE Felt Pledgets (Thick) | 007971 | 6.0 | 2.87 | 1/4 | 250 | Round |  |

PTFE Felt Pledgets are packed 10 per sterile pouch, 25 pouches per box.

BARD®

Cardiovascular Fabrics, Surgical Felts, Pledgets and Tapes

BARD® Cardiovascular Fabrics

Indications for Use:

1) The knitted and woven polyester fabrics (BARD® SAUVAGE® Filamentous fabrics, BARD® DEBAKEY® Double Velour fabrics, BARD® DEBAKEY® Elastic Knit fabrics, BARD® DEBAKEY® Woven fabrics) are indicated for use in cardiovascular surgical procedures requiring patch graft angioplasty such as carotid endarterectomy. These fabrics are also indicated for repair of certain intracardiac anomalies such as septal defects. **2)** DEBAKEY® Woven Fabrics and Edwards Outflow Tract Fabrics are indicated for aortic and pulmonary outflow tract repair and repair of intracardiac defects.

Contraindications:

Due to the high permeability of the knitted polyester fabrics, they are contraindicated for use in patients requiring prolonged systemic or high dose heparinization except for use in repair of intracardiac defects.

Warnings:

1) These fabrics must be properly preclotted with non-heparinized blood prior to exposure to full arterial pressure to avoid unnecessary bleeding or blood loss. Preclotting is not required when the fabrics are used in the intracardiac position. **2)** While it is common practice to employ moderate doses of intraoperative heparin, care should be taken not to exceed the manufacturer's recommended dose for such procedures. Excessive amounts of heparin can result in bleeding. **3)** Due to variability in patient response to heparinization, it is essential that the adequacy of anticoagulation during surgery and the precision of neutralization at the conclusion of the procedure be closely monitored. Adherence to a strict protocol, determined by each hospital, can prevent excessive bleeding. **4)** As with any cardiovascular fabric, occasional difficulties with hemostasis may occur. In the event that hemostasis is not easily obtained, the surgeon may wish to consider the following: • Investigate for systemic coagulopathy and treat appropriately. • Compression, as necessary. • Additional sutures and/or pledgets at the anastomosis, as necessary. • Reversal of heparin with protamine sulfate, as necessary. • Utilization of topical coagulation therapy, e.g., thrombin, as necessary. **5)** As with all woven fabric constructions, care must be taken when trimming the fabric to minimize the potential for fraying at cut edges. Caution is highly recommended for heat sealing on all woven patch edges. If the edges of the fabric are not heat sealed, then sutures must be at least 2 mm from the cut edge. **6)** Do NOT expose PTFE Fabrics to temperatures greater than 500°F (260°C). PTFE decomposes at elevated temperatures, producing highly toxic decomposition products.

Precautions:

1) STERILE, unless the package is opened or damaged. Single use only. **2)** These products are sterilized by ethylene oxide. DO NOT

RESTERILIZE. **3)** Care should be taken when clamping the fabrics to avoid damage to the fibers and preclot. Only atraumatic, shod-type vascular clamps should be used. **4)** Taper point, noncutting needles are recommended so the fabric fibers will not be cut during suturing. **5)** After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Potential Adverse Reactions:

Adverse reactions that may occur with the use of these products or with any cardiovascular implant procedure include: • Anastomotic aneurysms • Implant bleeding • Infection • Perioperative hemorrhage • Tissue erosion

BARD® Surgical Felts, Pledgets and Tapes

Indications for Use:

BARD® PTFE and Polyester Surgical Felts are used in various applications for general, vascular and cardiac surgery. They are most commonly used as a patch, a buttress for sutures, and as a material for replacement of segments of the ventricular myocardium after resection.

BARD® PTFE Felt Pledgets are used as buttresses under sutures when there is a possibility of sutures tearing through tissue. These pledgets are used in various surgical suturing procedures, such as vascular closure, septal repair, myocardial closure, hepatic repair and valvular suturing.

BARD® PTFE and Polyester Surgical Tapes are indicated for the purposes of tissue approximation, retraction, and as a temporary or permanent ligature.

Contraindications:

None known.

Warnings:

1) When used as a permanent ligature, care should be taken with surgical tapes to insure that the knots are secure. **2)** Do NOT expose PTFE products to temperatures greater than 500°F (260°C). PTFE decomposes at elevated temperatures, producing highly toxic decomposition products.

Precautions:

1) STERILE, unless the package is opened or damaged. Single use only. **2)** These products are sterilized by ethylene oxide. DO NOT RESTERILIZE. **3)** Care should be taken when clamping surgical tapes to avoid damage to the braided material. Only atraumatic instruments should be used. **4)** Taper point, noncutting needles are recommended

when surgical tapes are used as an implant material so the fibers will not be cut during suturing. **5)** Avoid excessive manipulation of the felt surface prior to use. **6)** To assure that felt fibers are cleanly cut and not pulled apart, it is important to use a sharp cutting device or scissors with sharp blades when cutting felt in to smaller pieces. **7)** After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Potential Complications:

Potential complications that may occur with the use of these products or with any vascular implant procedure include: • Infection • Tissue erosion

Please consult package insert for safety information and instructions for use.

The materials contained in this brochure are marketing samples only and are not to be used for human implant. See individual package instructions for more complete product information.

Bard and DeBakey are trademarks and/or registered trademarks of C. R. Bard, Inc.

SAUVAGE is a registered trademark licensed to C. R. Bard, Inc.

Copyright © 2014, C. R. Bard, Inc. All Rights Reserved.

S120073 Rev 1

BARD | PERIPHERAL
VASCULAR

Bard Peripheral Vascular, Inc.
1625 W. 3rd Street
Tempe, AZ 85281 USA
Tel: 1 480 894 9515 / 1 800 321 4254
Fax: 1 480 966 7062 / 1 800 440 5376
www.bardpv.com